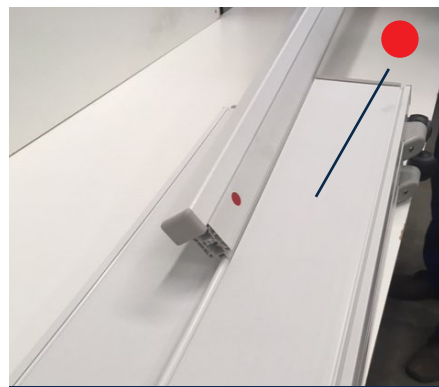
**BIOMBO STANDARD MODELO MÓVEL (2 bases)**  
 STANDARD FOLDING SCREEN MOBILE MODEL (2 bases)  
 BIOMBO STANDARD MODELO MÓBIL (2 bases)

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*The mounting procedure for the rod / stand assembly is the same as for the fixed model. The only difference is that one of the bases is wider and has a pre-installed lashing strap.*

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*Lay the screen joint horizontally, leaving the blade with red adhesive upwards (see photo).*

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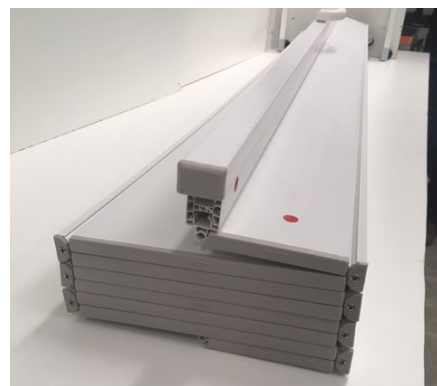
*Insert and slide the movable base onto the blade with the red sticker all the way (see photo).*

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*Gently lift the screen and complete the installation following the other manual contained in the kit.*

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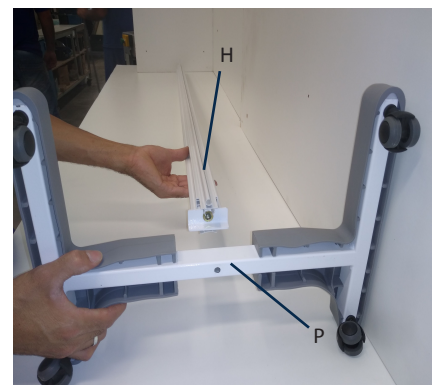
*For the mobile model (with 2 bases), repeat the procedure for the second base, following the demarcation of the blue stickers.*

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
*Align the rod (H) with the stand (P) and make sure that the smaller side of the rod is positioned downwards (see photo).*

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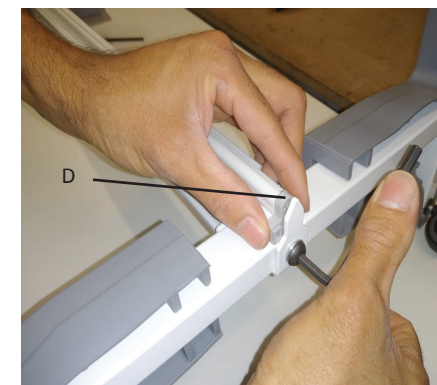
*Position the stem (D) shim between the hook profile and the tab and tighten the allen screw securely.*

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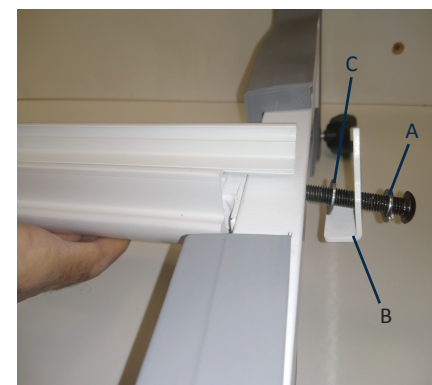
*Take the Allen screw and then insert the lock washer (A) first, then the tab (B), then the normal washer (C).*

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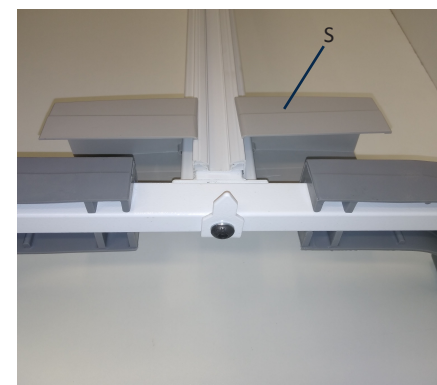
*Insert the top cover (S) of the Coupling Plus starting from the top of the rod to its bottom end.*

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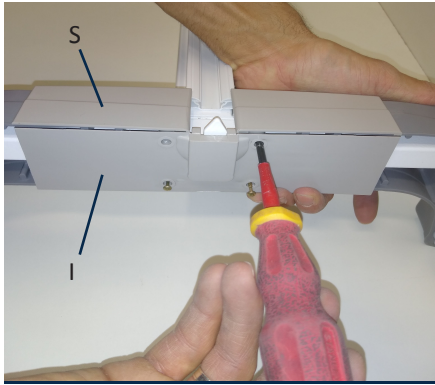*Screw the bottom cover (I) into the top cover (S) with the 4 Philips 3.0x30 screws.*

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*With the aid of the self tapping screw contained in the kit, drill the strap at the location marked in red.*

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*The mounting procedure for the rod / stand assembly is the same as for the fixed model. The only difference is that on the widest base a lashing strap must be attached (as the next steps).*

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*Attach the base to the plus screen joint (follow instructions on page 2).*

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*Screw the strap into the marked hole next to the red indicator on the rod pull profile.*

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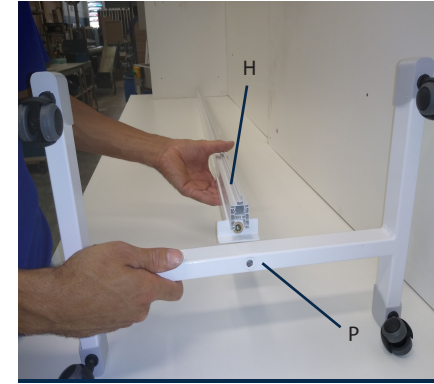
www.bcf.com.br

**BIOMBO STANDARD MODELO FIXO (1 base)**

STANDARD FOLDING SCREEN FIXED MODEL (1 base)

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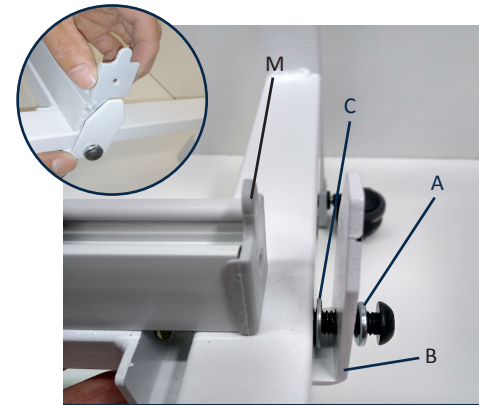
*Mount the coupling between the rod / stand assembly junction (see photo).*

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
*Fit the lid (M) on the jamb. Take the Allen screw and then insert the lock washer (A) first, then the tab (B), then the normal washer (C) and tighten the screw firmly.*

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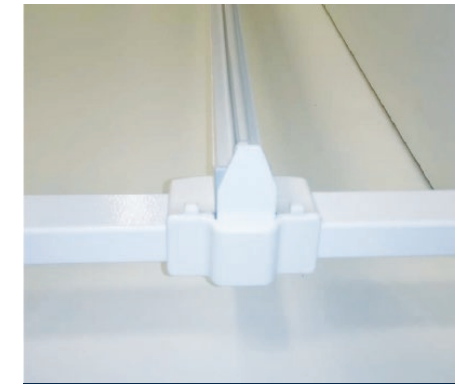*Press the coupling until fully seated.*

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