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HOSPITAL SCREEN ASSEMBLY / INSTALACIÓN DEL BIOMBO

5 Parafuse a tampa de acabamento no montante da parede.

Fasten the finishing cover on the jamb fixed on the wall.
Atornillar la tapa de acabamiento en la jamba de la pared.

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Place the second hook marked with a blue adhesive on the last panel of the hospital screen also marked with the blue adhesive. Observe the correct positioning of the adhesives according to the width of the hospital screen.

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To place the set of panels and hook on the jamb of the mobile base, get together the upper end of the hook on the upper part of the jamb in a way that the blue adhesives be aligned and firmly press and gradually from up to bottom until it totally fits.

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End the assembly placing the finishing cover on the mobile base, fasten the cover and insert the finishing pin over the screw, as shown in the above figure.

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For the installation of the mobile hospital screen of two bases, follow the steps of figures 6, 7 and 8; pay attention to the alignment to the corresponding adhesive.

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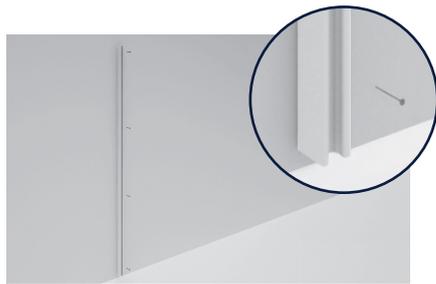
The jamb profile is bivalent, meaning that both sides are equal. Define the location and fasten the jamb profile on the wall. ATTENTION: The jamb of the wall is bigger than the one from the base.

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Make four holes in the jamb profile, lean it on the floor and fasten it on the wall with 4,2 x 3,8mm screws and 6mm plugs. If necessary, cut the jamb profile to fit on the baseboard.

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Insert the hook on the panel assuring that the red adhesives are on the same side and observing the image above, push it until the end.

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To place the set of panels and hook on the jamb fixed on the wall, get together the upper tip of the hook with the tip of the jamb, press firmly and gradually from top to bottom until it totally fits.

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Pay attention to the alignment of the corresponding adhesives:
Red adhesive ● with red adhesive ●
Blue Adhesive ● with blue adhesive ●

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Check our complete line of BCF products on our website:
www.bcf.com.br

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FOR MORE INFORMATION ABOUT THE ASSEMBLY,
WATCH OUR VIDEOS

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